Breezy® Elegance

Premium Retailer: This manual must be given to the user of this wheelchair.

User: Before using this wheelchair, read this entire manual and save it for future reference.

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**Manual de
Instrucciones**

WE LISTEN

Thank you for choosing a Breezy wheelchair. We want to hear your questions or comments about this manual, the safety and reliability of your chair, and the service you receive from your Premium Retailer. Please feel free to write or call us at the address and telephone number below:

Sunrise Medical
Customer Service Department
2842 Business Park Ave
Fresno, CA 93727
(800) 333-4000

Be sure to return your warranty card, and let us know if you change your address. This will allow us to keep you up to date with information about safety, new products and options to increase your use and enjoyment of this wheelchair. If you lose your warranty card, call or write and we will gladly send you a new one.

FOR ANSWERS TO YOUR QUESTIONS

Your Premium Retailer knows your wheelchair best, and can answer most of your questions about chair safety, use and maintenance. For future reference, fill in the following:

Premium Retailer: _____

Address: _____

Telephone: _____

Serial #: _____ Date/Purchased: _____

I. INTRODUCTION	2	H.Quick-Release Axles	12
II. TABLE OF CONTENTS	3	I. Push Handles	13
III. YOUR CHAIR AND ITS PARTS	4	J. Rear Wheels	13
IV. NOTICE - READ BEFORE USE	5	K.Drum Brake (Gold and Platinum models)	13
A.Choose the Right Chair & Safety Options	5	L. Rear Wheel Locks	13
B.Review This Manual Often	5	M. Modified Seat Systems	13
C.Warnings	5	N.Seat and Back Upholstery	13
V. GENERAL WARNINGS.....	5	VIII. OWNER USE AND MAINTENANCE.....	14
A.Weight Limits	5	A.Introduction.....	14
B.Intended Use	5	B. Critical Maintenance Tips	14
C.Attendants and Caregivers	5	C.Maintenance Chart	14
D.Accessories	5	D.Cleaning.....	14
E. Know Your Chair	5	E. Troubleshooting Chart.....	14
F. Reduce The Risk of an Accident	6	F. To Mount and Remove Rear Wheels	15
G.Safety Checklist	6	G.Wheel Locks	15
H.Changes & Adjustments	6	H.Drum Brake.....	15
I. Environmental Conditions	6	I. Anti-tip tubes (Optional).....	16
J. Terrain	6	J. Flip Back Armrests	16
K.Street Use	6	K.Flip-Back Height-Adjustable Armrests	
L. Motor Vehicle Safety	6	(Gold and Platinum models).....	16
M.When You Need Help	6-7	L. Swing-in/Swing out Footrests	16
VI. SAFETY WARNINGS: FALLS & TIP-OVERS.....	7	M.Elevating Legrest (Optional).....	17
A.Center of Balance	7	N.Tension adjustable Backrest	17
B. Dressing or Changing Clothes	7	O.Standard or Height Adjustable Push Handle	
C.Obstacles	7	(Gold and Platinum models).....	17
D.Front Caster Lift.....	7	P. Cushion Care and Maintenance	17
E. Reaching or Leaning	8	Q.To Unfold or Fold your Wheelchair	17
F. Moving Backward	8	R.Check-Out.....	18
G.Escalators.....	8	IX. PREMIUM RETAILER	
H.Ramps, Slopes & Sidehills	8	SERVICE AND ADJUSTMENT.....	19
I. Transfer.....	9	A.Premium Retailer Service Introduction	19
J. Curbs & Single Steps	10	B. Critical Maintenance tips.....	19
K.Climbing a Curb or Single Step.....	10	C.Cleaning.....	19
L. Descending a Curb or Single Step.....	10	D.Rear Wheel Axle Nut Adjustment	19
M.Stairs	11	E. Vertical Axle Position	19
N.Climbing Stairs	11	F. Swing-in/Swing-out Footrest.....	20
O.Descending Stairs	11	G.Caster Axle Angle Adjustment.....	20
VII. WARNINGS: COMPONENTS & OPTIONS	12	H.Wheel Lock Adjustment	20
A.Anti-Tip Tubes (Optional)	12	I. Length Adjustable Armrest Pads	21
B. Armrests	12	J. Tension Adjustable Upholstery	21
C.Cushions & Sling Seats	12	K.Standard and Low Front Seat Heights	21
D.Fasteners	12	L. Seat Sling Replacement	21
E. Footrests and Legrests.....	12	M.Back Depth Adjustment	22
F. Unfolding and Folding	12	N.Check-Out	22
G. Polyurethane Tires.....	12	X. SUNRISE LIMITED WARRANTY	23
		ESPAÑOL	24

1. Push Handle/Hub Locks
2. Tension Adjustable Backrest Upholstery
3. Cushion
4. Hanger/Footrest
5. Caster Wheel
6. Footplates
7. Folding Frame
8. Caster Fork
9. Rear Wheel
10. Handrim
11. Wheel Lock
12. Adjustable Flip-back Armrest
13. Adjustable Armrest Pad

BREEZY ELEGANCE

Weight

33 to 38 lbs.

Frame Dimensions

Frame width: 17" - 19"

(seat width 1/2" narrower)

Seated depth: 16" - 18"

Seat to floor height: 16.7" - 20"

Backrest

80° with push handles

Height: 16" - 18"

Hanger Type

Standard - 70° Swing away

Optional - 90° elevating Legrest

Footplates

Composite, adult angle-adjustable with heel loops.

Casters

Caster wheel - 8" polyurethane

3/4" longer fork stem bolt.

Armrests

Standard - Flip-back with adjustable pad.

Optional - Flip-back, height-adjustable with adjustable pad

Axles

Standard - Quick -release

Anti-tip Tubes

Optional - Rear anti-tip tubes

Standard Rear Wheels

Standard - spoke

Tire Size

Standard - 24"

Tire types

Standard - Solid type with spoked wheels

Handrims

Standard - aluminum

Wheel Locks

Standard - high-push

6" extension handles

Cushions

Silver Cushion

Gold Cushion

Platinum Cushion

All features may not be available with some chair setups or in conjunction with another chair feature. Please consult your Premium Retailer for more information. Your Premium Retailer can also provide you with more information on accessories for your purchase.

A. CHOOSE THE RIGHT CHAIR & SAFETY OPTIONS

Sunrise provides a choice of many wheelchair styles to meet your needs. This product is intended for single person use only. Final selection of the type of wheelchair, options and adjustments rests solely with you and your Healthcare advisor. Choosing the best chair for your unique mobility needs, will depend on such things as:

1. Your disability, strength, balance and coordination.
2. The types of hazards you must overcome in daily use (where you live and work, and other places you are likely to use your chair).
3. The need for options that will improve your positioning, safety and comfort (such as anti-tip tubes, positioning belts, or special seating systems).

B. REVIEW THIS MANUAL OFTEN

Before using this chair you, and each person who may assist you, should read this entire manual and make sure to follow all instructions. Review the warnings often, until they are second nature to you.

C. WARNINGS

The word “WARNING” refers to a hazard or unsafe practice that may cause severe injury or death to you or to other persons. The “Warnings” are in three main sections, as follows:

1. V — GENERAL WARNINGS

Here you will find a safety checklist and a summary of risks you need to be aware of before you ride this chair.

2. VI— SAFETY WARNINGS: FALLS & TIP-OVERS

Here you will learn about practices for the safe use of your chair, and how to avoid a fall or tip-over while you perform daily activities in your chair.

3. VII — WARNINGS — COMPONENTS & OPTIONS

Here you will learn about the components of your chair and options you can select for safety. Consult your Premium Retailer and your health care advisor to help you choose the best set-up and options for safe use.

NOTE— Where they apply, you will also find “Warnings” in other sections. Heed all warnings in these sections. If you fail to do so, a fall, tip-over or loss of control may occur and cause severe injury to the rider and/or others.

V. GENERAL WARNINGS

A. WEIGHT LIMITS **WARNING**

NEVER exceed the weight limit specified by Sunrise Medical. The weight capacity provided by your manufacturer is for the **combined** weight of a rider and items carried. If you do exceed the weight limit, damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

Weight Capacity

Breezy Elegance - 275lbs (125kg.)

B. INTENDED USE

The Breezy Elegance Series of wheelchair's intended use is to provide mobility to persons limited to a sitting position.

 WARNING

DO NOT use this device for purposes other than what is intended by the manufacturer

1. The wheelchair is not designed for weight training and is unsafe for use as a seat while weight training. Weight training from the wheelchair substantially changes the stability of the chair and may cause tipping.
2. **DO NOT** stand on the frame of the wheelchair.
3. **NEVER** allow someone to stand on your chair or use it as a step ladder.
4. **This chair is designed for a single rider only.**
5. Unauthorized modifications and use of parts or accessories not supplied or approved by Sunrise Medical may change the chair structure. This will void the warranty and may cause a safety hazard. If the warning is ignored, damage to your chair, and the potential for severe injury of the person using the chair for unintended purposes can occur.

C. ATTENDANTS AND CAREGIVERS **WARNING**

Before you assist a rider, be sure to read all warnings contained in this manual, and follow all instructions that apply. Be aware that after consulting a healthcare advisor, you will need to learn safe and proven body mechanics to use and create assistive methods best suited to your abilities.

D. ACCESSORIES **WARNING**

Unauthorized modifications or use of parts, or accessories not supplied or approved by Sunrise Medical may change the chair structure. This will void the warranty and may cause a safety hazard.

Some problems that may include, but are not limited to:

1. Incorrect Wheels and/or tires that put the rider at risk of a fall or tip-over.
2. Adding a component to the frame, changing the structural integrity of the chair.
3. Modification or disassembly can potentially create an unsafe situation where rider and/or attendant are put at risk.

E. KNOW YOUR CHAIR **WARNING**

Every wheelchair is different. Take the time to learn the feel of this chair before you begin riding. Start slowly, with easy, smooth strokes. If you are used to a different chair, you may use too much force and tip over. If you use too much force, damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

F. REDUCE THE RISK OF AN ACCIDENT

⚠ WARNING

1. **BEFORE** you begin riding, you should be trained in the safe use of this chair by your Premium Retailer.
2. Practice bending, reaching and transfers until you know the limit of your ability. Have someone help you until you know what can cause a fall or tip-over and how to avoid doing so.
3. Be aware that you must develop your own methods for safe use best suited to your level of function and ability.
4. **NEVER** try a new maneuver on your own. Get advice from your Premium Retailer to lower the risk of a fall or tip over.
5. Get to know the areas where you plan to use your chair. Look for hazards and learn how to avoid them.

G. SAFETY CHECKLIST

⚠ WARNING

Before Each Use Of Your Chair:

1. Make sure the chair rolls easily and that all parts work smoothly. Check for noise, vibration, or a change in ease of use. (This may indicate damaged tires, loose fasteners, or damage to your chair).
2. Immediately repair any problem. Your Premium Retailer can help you find and correct the problem.
3. Check to see that both quick-release rear axles are locked. When locked, the axle button will "pop out" fully. If not locked, the wheel may come off and cause you to fall.
4. If your chair has anti-tip tubes, lock them in place.

H. CHANGES & ADJUSTMENTS

⚠ WARNING

1. See your Premium Retailer when adjustment of seating or components needs to be made.
2. Unauthorized modifications or use of parts not supplied or approved by Sunrise may change the chair structure. This will void the warranty and may cause a safety hazard.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

I. ENVIRONMENTAL CONDITIONS

⚠ WARNING

1. Use extra care if you must ride your chair on a wet or slick surface. If you are ever in doubt as to your ability to navigate across wet or slick surfaces, ask for help.
2. Contact with water or excess moisture may cause your chair to rust or corrode. Avoid all extreme weather situations if possible.
 - a. **Do not** use your chair in a shower, pool or other body of water. The chair tubing and parts are not water-tight and may rust or corrode from the inside.
 - b. **Avoid** excess moisture (for example, do not leave your chair in a damp bathroom while taking a shower).
 - c. Dry your chair as soon as you can if it gets wet, or if you use water to clean it.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

J. TERRAIN

⚠ WARNING

1. Based on ANSI/RESNA testing, Sunrise Medical recommends the use of a caster wheel with a minimum diameter of 8", if the wheelchair will be overcoming obstacles up to 1/2" on a regular basis.
2. Your chair is designed for use on firm, even surfaces such as concrete, asphalt, indoor flooring, and carpets.
3. Do not operate your chair in sand, loose soil, or over rocky terrain.
4. If you use your chair on terrain that is rougher than described above there is a danger that screws and bolts will loosen prematurely, and that damage to wheels or axles could put the rider at risk of a fall, tip-over, or loss of control.

K. STREET USE

⚠ WARNING

1. This product is not intended for street use.
2. Avoid streets whenever possible.
3. Obey and follow all legal pedestrian pathways, and laws that apply to pedestrians.
4. Be alert to the danger of motor vehicles in parking lots, or if you must cross a road.

If you fail to heed this warning, damage to your chair, and a possible collision may occur and cause severe injury to the rider or others.

L. MOTOR VEHICLE SAFETY

The rider of this chair should transfer to the Original Equipment Manufacturer vehicle seat and use the OEM vehicle restraint.

⚠ WARNING

Your chair is **NOT** equipped with a Transit Option:

1. **NEVER** let anyone sit in this chair while in a moving vehicle.
 - a. **ALWAYS** move the rider to an approved vehicle seat.
 - b. **ALWAYS** secure the rider with proper motor vehicle restraints.
2. In an accident or sudden stop the rider may be thrown from the chair. Wheelchair seat belts will not prevent this, and further injury may result from the belts or straps.
3. **NEVER** transport this chair in the front seat of a vehicle. It may shift and interfere with the driver.
4. **ALWAYS** secure this chair so that it cannot roll or shift.
5. Do not use any chair that has been involved in a motor vehicle accident. The frame and/or components may have been changed due to the accident. Such items could be, but are not limited to: bent, loosened, and/or broken components that were subjected to an impact.

Failure to heed these warnings puts you and others that may assist you at a high risk of injury.

M. WHEN YOU NEED HELP

⚠ WARNING

For The Rider: Make sure that each person who helps you reads and follows all warnings and instructions that apply.

For Attendants:

1. Work with the rider's Premium Retailer to learn safe methods best suited to your abilities and those of the rider.
2. Always tell the rider what you plan to do, and explain what you expect the rider to do. This will put the rider at ease and reduce the risk of an accident.

3. This chair has push handles. They provide secure points for you to hold the rear of the chair to prevent a fall or tip-over. Check to make sure push handle grips will not rotate or slip off.
 4. To prevent injury to your back, use good posture and proper body mechanics. When you lift or support the rider or tilt the chair, bend your knees slightly and keep your back as upright and straight as you can.
 5. Remind the rider to lean back when you tilt the chair backward.
 6. When you descend a curb or single step, slowly lower the chair in one easy movement. Do not let the chair drop the last few inches to the ground. This may damage the chair or injure the rider.
 7. To avoid tripping, unlock and rotate anti-tip tubes up, out of the way.
 8. Whenever you aren't attending the wheelchair, **ALWAYS** use the Hub lock, and wheel-lock to secure the rear wheels, and lock the anti-tip tubes in place.
- If you fail to ask for help you run a high risk of a fall, tip-over or loss of control that may occur and cause severe injury to the rider or others.

VI. SAFETY WARNINGS: FALLS & TIP-OVERS

A. CENTER OF BALANCE

WARNING

The point where this chair will tip forward, back or to the side depends on its center of balance and stability. How your chair is set up, the options you select and the changes you make may affect the risk of a fall or tip-over.

1. The center of balance is also affected by:
 - a. A change in the set-up of your chair, including:
 - The distance between the rear wheels.
 - The amount of rear wheel camber.
 - The seat height and seat angle.
 - Backrest angle.
 - b. A change in your body position, posture or weight distribution.
 - c. Riding your chair on a ramp or slope.
 - d. A back pack or other options and the amount of added weight.
2. To Reduce The Risk Of An Accident:
 - a. Consult your healthcare advisor to find out what axle and caster position is best for you.
 - b. Consult your Premium Retailer **BEFORE** you modify or adjust this chair. Be aware that you may need to make other changes to correct the center of balance.
 - c. Have your Premium Retailer help you until you know the balance points of your chair and how to avoid a tip-over.

If you fail to heed these warnings, you are at a high risk of a fall, tip-over or loss of control that could cause severe injury to the rider or others.

B. DRESSING OR CHANGING CLOTHES

WARNING

Your weight may shift if you dress or change clothes while seated in this chair.

To reduce the risk of a fall or tip-over:

1. Rotate the front casters until they face forward. This makes the chair more stable.
2. Lock anti-tip tubes in place. If your chair does not have anti-tip tubes, back it up against a wall and lock both rear wheels.

If you fail to heed this warning, damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

C. OBSTACLES

WARNING

Riding over curbs or obstacles can cause tipping and serious bodily harm. If you have any doubt that you can safely cross any curb or obstacle, **ALWAYS ASK FOR HELP**. Be aware of your riding skills and personal limitations. Develop new skills only with the help of a companion.

To avoid the risk of traversing obstacles:

1. Keep a lookout for danger – scan the area well ahead of your chair as you ride.
2. Make sure the floor areas where you live and work are level and free of obstructions.
3. Remove or cover threshold strips between rooms.
4. Install a ramp at entry or exit doors. Make sure there is not a drop off at the bottom of the ramp.
5. To Help Correct Your Center Of Balance:
 - a. Lean your upper body **FORWARD** slightly as you go **UP** over an obstacle.
 - b. Press your upper body **BACKWARD** as you go **DOWN** from a higher to a lower level.
6. If your chair has anti-tip tubes, do not go over an obstacle without help.
7. Keep both of your hands on the handrims as you go over an obstacle.
8. Never push or pull on an object (such as furniture or a doorjamb) to propel your chair.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

D. FRONT CASTER LIFT

WARNING

Front caster lift can occur when the front wheels, ordinarily in contact with the ground, are either intentionally, or unintentionally caused to lift from the ground while the rear wheels remain in contact. Pitch control (partial or full) should **NEVER** be attempted without consulting your Premium Retailer and making sure that anti-tips are installed.

If you fail to heed this warning, you are at a high risk of damage to your chair, a fall, tip-over or loss of control that may occur and cause severe injury to the rider or others.

E. REACHING OR LEANING

⚠ WARNING

If you reach or lean it will affect the center of balance of your chair. This may cause you to fall or tip over. When in doubt, ask for help or use a device to extend your reach.

1. **NEVER** reach or lean if you must shift your weight sideways or rise up off the seat.
2. **NEVER** reach or lean if you must move forward in your seat to do so. Always keep your buttocks in contact with the backrest.
3. **NEVER** reach with both hands (you may not be able to catch yourself to prevent a fall if the chair tips).
4. **NEVER** reach or lean to the rear unless your chair has anti-tip tubes locked in place.
5. **DO NOT** reach or lean over the top of the seat back. This may damage one or both backrest tubes and cause you to fall.
6. If You Must Reach Or Lean:
 - a. **Do not** lock the rear wheels. This creates a tip point and makes a fall or tip-over more likely.
 - b. **Do not** put pressure on the footrests.
 - c. Move your chair as close as you can to the object you wish to reach.
 - d. **Do not** try to pick up an object from the floor by reaching down between your knees. You are less likely to tip if you reach to the side of your chair.
 - e. Rotate the front casters until they are as far forward as possible. This makes the chair more stable.
 - f. Firmly grasp a rear wheel or an armrest with one hand. This will help to prevent a fall if the chair tips.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

F. MOVING BACKWARD

⚠ WARNING

Use extra care when moving your chair backward. Your chair is most stable when you propel yourself forward. You may lose control or tip over if one of the rear wheels hits an object and stops rolling while moving backward.

1. Propel your chair slowly and smoothly.
2. If your chair has anti-tip tubes, make sure to lock them in place.
3. Stop often and check to be sure your path is clear.

If you fail to heed this warning, you are at a high risk of damage to your chair, a fall, tip-over or loss of control that may occur and cause severe injury to the rider or others.

G. ESCALATORS

⚠ WARNING

NEVER use this chair on an escalator, even with an attendant. If you do, a fall or tip-over is likely.

If you fail to heed this warning you and others that may be assisting you, are at a high risk of severe injury.


H. RAMPS, SLOPES & SIDEHILLS 1 2 3 4

⚠ WARNING

Whenever possible, avoid riding on a slope, which includes a ramp or sidehill. This will change the center of balance of your chair. Your chair is less stable and more difficult to maneuver when it is at an angle. When moving up a hill, anti-tip tubes may not prevent a fall or tip-over. Always ask for help when riding on slopes.


1. **Do not** use your chair on a slope steeper than 10°.
2. Avoid Sidehills. **Do not** use your chair on a sidehill with a slope steeper than 6°.
3. **Always** go straight up or straight down a slope. **Do not** "cut the corner" on a slope or ramp.
4. **Do not** turn or change direction on a slope.
5. **Do not** stop on a steep slope. If you stop, you may lose control of your chair.
6. **NEVER** use rear wheel locks to try to slow or stop your chair. This is likely to cause your chair to veer out of control.
7. **ALWAYS** be aware of:
 - a. Wet or slippery surfaces.
 - b. A change in grade on a slope (or a lip, bump or depression). These may cause a fall or tip-over.
 - c. A drop-off at the bottom of a slope or ramp. A drop-off as small as 3/4 inch can stop a front caster and cause the chair to tip forward.
8. To Reduce The Risk Of A Fall Or Tip-Over:
 - a. Lean or press your body UPHILL. This will help adjust for the change in the center of balance caused by the slope or sidehill.
 - b. Keep pressure on the handrims to control your speed on a down slope. If you go too fast you may lose control.
9. Ramps at home & work— For your safety, ramps at home and work must meet all legal requirements for your area:
 - a. Avoid a drop off. You may need a section at the top or bottom to smooth out the transition.
 - b. Stay in the center of the ramp and control your speed.
10. When you have to use a ramp, always stay in the CENTER of the ramp. Make sure the ramp is wide enough so that you have no risk of going off the edge.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and can possibly cause severe injury to the rider or others.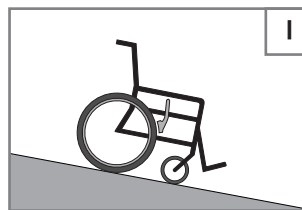


Downslope: less than 10°

- Go straight down
- Control your speed
- Go Slow
- No Turns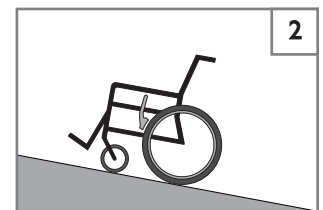


Upslope: less than 10°

- Lean Forward
- Sit straight up
- Don't stop
- No Turns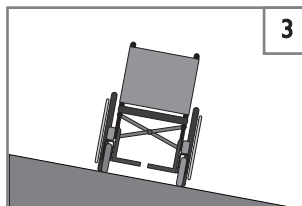

Sidehill: less than 6°

- Don't stop
- No Turns
- Lean uphill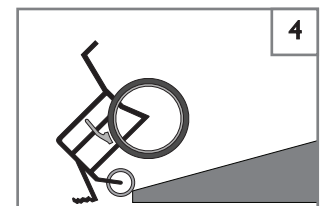

Ramps

- No Drop-offs
- Go straight down
- Stay centered, Go Slow

I. TRANSFERS 5 6**⚠ WARNING**

It is dangerous to transfer on your own. It requires good balance and agility. Be aware that there is a point during every transfer when the wheelchair seat is not below you.

To avoid a fall:

1. Work with your Premium Retailer to learn safe methods for transfers.
 - a. Learn how to position your body and how to support yourself during a transfer.
 - b. Have someone help you until you learn safe transfer methods.
2. Lock the rear wheels before you transfer.
3. Be aware that the chair can still slide and/or tip. The wheel lock keeps the rear wheels from rolling while you are performing the transfer.
4. Make sure that the tires are in good shape. damaged tires may allow the rear wheel locks to slip. (see table in Section G “Pneumatic Tires”)


Transferring:

1. Move your chair as close as you can to the seat you are transferring to. If possible, use a transfer board.
2. Rotate the front casters until they are as far forward as possible.
3. Set the wheel lock and/or Drum Brake.
4. If you can, remove the footrests, and/or swing them out of the way.
 - a. Make sure your feet do not catch in the space between the footrests.
 - b. Avoid putting weight on the footrests as this may cause the chair to tip.
5. Make sure armrests are removed, or out of the way and do not interfere with the transfer.
6. Transfer as far back onto the seat surface as you can. This will reduce the risk that the chair will tip or move away from you.

Optimum Transfer position

**Before transferring:**

1. Remove the armrests.
2. Remove or move the footrests.
3. Make sure the wheel lock is set.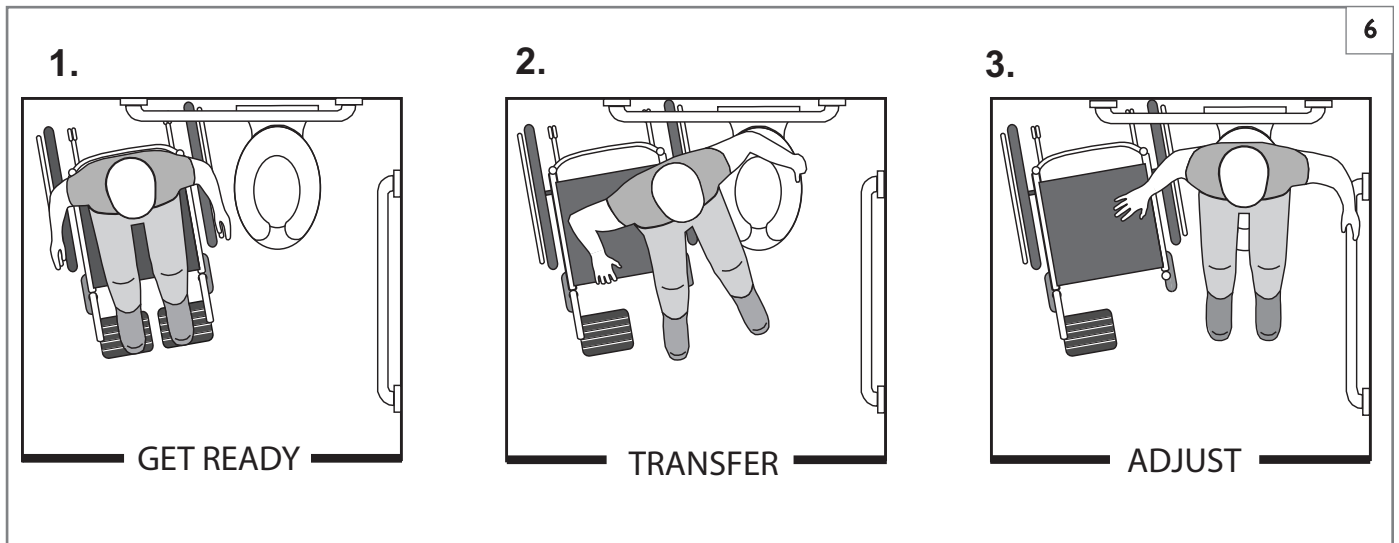

J. CURBS & SINGLE STEPS**⚠ WARNING**


Before riding over curbs, and negotiating even a single step, **ALWAYS** ask for assistance first. Curbs and steps can cause tipping and serious bodily harm. When in doubt as to your ability to avoid, or traverse any obstacle, always ask for help. Be aware of your riding skills and personal limitations. Develop new skills only with the help of a companion.

For Attendant::

Each person who assists the rider with curbs and steps should read and follow all instructions and warnings pertaining to attendants, and caregivers.

1. Do not try to climb a high curb or step (more than 4 inches high) **UNLESS** you have help. Doing so may cause your chair to exceed its balance point and tip over.
2. Go straight up and straight down a curb or step. If you climb or descend at an angle, a fall or tip-over is likely.
3. Be aware that the impact of dropping down from a curb or step can damage your chair or loosen fasteners.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and can possibly cause severe injury to the rider or others.

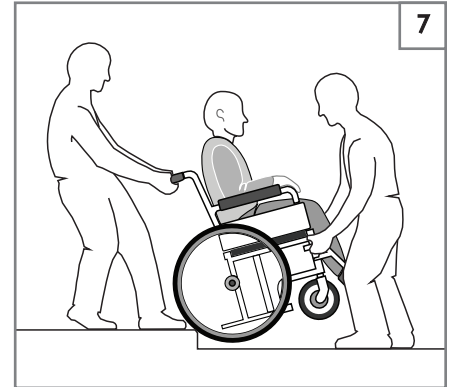**K. CLIMBING A CURB OR SINGLE STEP****⚠ WARNING****For Attendant::**

Follow these steps to help the rider climb a curb or single step going **FORWARD**:

1. Stay behind the chair.
2. Face the curb and tilt the chair up on the rear wheels so that the front casters clear the curb or step.
3. Move forward, placing the front casters on the upper level as soon as you are sure they are past the edge.
4. Continue forward until the rear wheels contact the face of the curb or step. Lift and roll the rear wheels to the upper level.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

L. DESCENDING A CURB OR SINGLE STEP**⚠ WARNING****For Attendant::**

Follow these steps to help a rider descend a curb or single step going **BACKWARD**:

1. Stay at the rear of the chair.
2. Several feet before you reach the edge of the curb or step, turn the chair around and pull it backward.
3. While looking over your shoulder, carefully step back until you are off the curb or stair and standing on the lower level.
4. Pull the chair toward you until the rear wheels reach the edge of the curb or step. Then allow the rear wheels to slowly roll down onto the lower level.
5. When the rear wheels are safely on the lower level, tilt the chair back to its balance point. This will lift the front casters off the curb or step.
6. Keep the chair in balance and take small steps backward. Turn the chair around and gently lower front casters to the ground.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

M. STAIRS**⚠ WARNING**

1. **NEVER** use this chair on stairs **UNLESS** you have someone to help you. Doing so is almost certain to cause a fall or tip-over.
2. Negotiating stairs with a wheelchair always requires at least 2 attendants for safety.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

N. CLIMBING STAIRS**⚠ WARNING****For Attendants::**


1. Use at least two attendants to move a chair and rider up stairs.
2. Move the chair and rider **BACKWARD** up the stairs.
3. The person at the rear is in control. He or she tilts the chair back to its balance point.
4. A second attendant at the front firmly grasps a non-detachable part of the front frame and lifts the chair up and over one stair at a time.
5. The attendants move to the next stair up. Repeat for each stair, until you reach the landing.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

O. DESCENDING STAIRS**⚠ WARNING****For Attendants::**

1. Use at least two attendants to move a chair and rider down stairs.
2. Move the chair and rider **FORWARD** down the stairs.
3. The person at the rear is in control. He or she tilts the chair to the balance point of the rear wheels and rolls it to the edge of the top step.
4. A second attendant stands on the third step from the top and grasps the chair frame. He or she lowers the chair one step at a time by letting the rear wheels roll over the stair edge.
5. The attendants move to the next stair down. Repeat for each stair, until you reach the landing or bottom of stairs.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.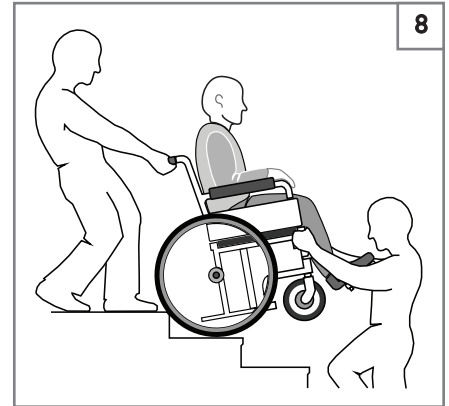

A. ANTI-TIP TUBES (OPTIONAL)**⚠ WARNING**

Anti-tip tubes can help keep your chair from tipping over backward in normal conditions.

1. Sunrise recommends the use of anti-tip tubes:
2. When locked in place (in the “down” position) anti-tip tubes should be **BETWEEN** 1 1/2 to 2 inches off the ground.
 - a. If set **higher** than 2 inches, they may not prevent a tip-over.
 - b. If set **lower** than 2 inches, they may “hang up” on obstacles and cause a fall or tip over.
3. If you have to climb or descend a curb, or overcome an obstacle it may be necessary to have an attendant make sure the anti-tip tubes are rotated up, and out of the way, so that the chair and rider do not get stuck and/or become unstable.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

B. ARMRESTS**⚠ WARNING**

Armrests can detach and will not bear the weight of this chair.

1. **NEVER** lift this chair by its armrests. They may come loose or break.
2. Lift this chair only by non-detachable parts of the main frame.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

C. CUSHIONS & SLING SEATS**⚠ WARNING**

1. Manual sling seats and standard foam cushions are not designed for the relief of pressure.
2. If you suffer from pressure sores or if you are at risk that they will occur, you may need a special seating system or a device to control your posture. Consult your healthcare advisor to find out if you need such a device for your well-being.
3. Seat slings are not intended to be used as a direct seating surface. A cushion or other seating surface should be placed on the sling before use.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

D. FASTENERS**⚠ WARNING**

Many of the screws, bolts and nuts on this chair are special high-strength fasteners. Use of improper fasteners may cause your chair to fail.

1. **ONLY** use fasteners provided by an authorized supplier (or ones of the same type and strength, as indicated by the markings on the heads).
2. Over- or under-tightened fasteners may fail or cause damage to chair parts.
3. If bolts or screws become loose, tighten them as soon as you can.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

E. FOOTRESTS & LEGRESTS

If it is necessary to adjust the length of your footrests remember to follow the following instructions for user safety.

⚠ WARNING

1. At the lowest point, footrests should be **AT LEAST** 2 inches off the ground. If set too **LOW**, they may “hang up” on obstacles you can expect to find in normal use. This may cause the chair to stop suddenly and tip forward.
2. To Avoid A Trip Or Fall When You Transfer:
 - a. Make sure your feet do not “hang up” or get caught in the space between the footrests.
 - b. Avoid putting weight on the footrests, as the chair may tip forward.
 - c. Footrests should be swung out of the way or removed whenever entering or exiting the wheelchair.
3. **NEVER** lift this chair by the footrests. Footrests detach and will not bear the weight of this chair. Lift this chair only by non-detachable parts of the main frame.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

F. UNFOLDING & FOLDING**⚠ WARNING**

1. To fully unfold your chair, place your palms on the seat rails located on each side of the seat sling and push down firmly until the rails are seated, and locked.
2. To fold your chair, first make sure to loosen the straps in your tension adjustable upholstery, and either remove the footrests, or fold up the footplates.
3. Standing in front of the chair, lift up on the center of the seat sling and then push both sides of the frame together.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

G. POLYURETHANE TIRES**⚠ WARNING**

As with any vehicle, the wheels and tires should be checked periodically for cracks and wear that could cause issues with balance and/or tracking.

1. Inspect your tires for flat spots and excessive wear, caused by drying and cracking.
2. If you do find flat spots, crack, or pits larger than 1/4” be sure to replace the tires as soon as possible.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

H. QUICK RELEASE AXLES**⚠ WARNING**

1. Do not use this chair **UNLESS** you are sure that both quick-release rear axles are locked.
2. An axle is not locked until the quick-release button pops out fully. If the axle is not inserted fully, the wheel may come off during use, endangering the rider.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

I. PUSH HANDLES**⚠ WARNING**

When you have an attendant, make sure they use the push handles.

1. Push handles provide secure points for an attendant to hold the rear of this chair, to prevent a fall or tip-over.
2. Check to make sure push handle grips will not rotate or slip off.
3. Make sure that the Adjustment lever is tight and that the Push handle tubes are not loose before any use of the chair.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

J. REAR WHEELS**⚠ WARNING**

A change in set-up of the rear wheels will affect the center of balance of your chair.

1. The farther you move the rear axles FORWARD, the more likely it is that your chair will tip over backward.
2. Consult your premium retailer to find the best rear axle set-up for your chair. Do not change the set-up **UNLESS** you consult your premium retailer first.
3. Adjust the rear wheel locks after you make any change to the rear axles.
 - a. If you fail to do so, the wheel locks may not work.
 - b. Make sure lock arms embed in the tires at least 1/8 inch when locked.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

K. DRUM BRAKE (GOLD AND PLATINUM MODELS)**⚠ WARNING**

When you have an attendant, make sure they brake and lock the drum brakes evenly. Uneven pressure on the drum brake levers can cause uneven wear of the brake components.

1. Drum brakes can be used by an attendant to control the speed, or lock the rear wheels of your chair.
2. When using the drum brakes be sure to squeeze or lock the two sides evenly. Applying too much pressure to one side can prematurely wear the mechanism, and cause uneven braking.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

L. REAR WHEEL LOCKS**⚠ WARNING**

Rear wheel locks are NOT designed to slow or stop a moving wheelchair. Use them only to keep the rear wheels from rolling when your chair is at a complete stop.

1. NEVER use rear wheel locks to try to slow or stop your chair when it is moving. Doing so may cause a fall or tip-over.
2. To keep the rear wheels from rolling, always set both rear wheel locks when you transfer to or from your chair.
4. Make sure lock arms embed in tires at least 1/8 inch when locked. If you fail to do so, the locks may not work.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

M. MODIFIED SEAT SYSTEMS**⚠ WARNING**

Use of a seating system not approved by Sunrise may alter the center of balance of your chair. This may cause the chair to tip over.

1. Do not change the seating system of your chair **UNLESS** you consult your Premium Retailer first.
2. Use of a seating system not approved by your Premium Retailer may affect the folding mechanism of this chair.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

N. SEAT AND BACK UPHOLSTERY**⚠ WARNING**

1. Replace worn or torn fabric on your seat and/or back as soon as you can. If you fail to do so, the seat or back may fail.
2. Sling fabric will weaken with age and use. Look for fraying, thin spots, or stretching of fabrics especially at edges and seams.
3. "Dropping down" into your chair will weaken fabric and result in the need to inspect and replace the seat more often.
4. Be aware that laundering or excess moisture will reduce the flame retardation qualities of the fabric that is provided with your chair.
5. If Tension Adjustable upholstery is present, loosen straps or remove upholstery prior to folding or unfolding the chair.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

⚠ WARNING

The owner of this chair is responsible for making sure that it has been setup and adjusted by a trained Premium Retailer under the advice of a healthcare advisor. The chair may require periodic maintenance or certain in-use adjustments that may be performed by the owner or caregiver.

A. INTRODUCTION

1. Proper maintenance will improve performance and extend the useful life of your chair.
2. Clean your chair regularly. This will help you find loose or worn parts and make your chair easier to use. You will need a mild detergent solution and plenty of cleaning rags.
3. Repair or replace loose, worn, bent or damaged parts **before using the chair**.
4. To protect your investment, have all major maintenance and repair work done by your Premium Retailer
5. **Inspect and maintain this chair strictly per the maintenance chart.**
6. **If you detect a problem, make sure to order parts, or have service, and repair work done at your Premium Retailer before use.**
7. **At least once per year, have a complete inspection, safety check, and service of your chair made by an Premium Retailer.**

B. CRITICAL MAINTENANCE TIPS

1. Axles & Axle Sleeves:

When checking axles and axle sleeves every six months, make sure they are clean and tight. Loose sleeves will damage the axle plate and will affect performance.

C. MAINTENANCE CHART

Weekly	3 Months	6 Months	Annually	Check...
✓				Tires for excessive wear, broken spokes
✓				Wheel locks engage 1/8" into tire
	✓			All fasteners for wear and tightness
	✓			Quick-release axles should lock
	✓			Caster tires for excessive wear
	✓			Anti-tip tubes for tightness
	✓			Upholstery for sagging and wear
		✓		Armrests for wear and tightness
		✓		Drum Brakes should lock wheels
		✓		Footrests for wear and tightness
		✓		Frame and Crossbrace hardware
			✓	Service by Premium Retailer

D. CLEANING

1. Paint Finish

- a. Clean the painted surfaces with mild soap or detergent at least once a month.
- b. Protect the paint with a coat of non-abrasive auto wax every three months.

2. Axles, Drum Brakes and all Moving Parts

- a. Clean around axles and moving parts WEEKLY with a slightly damp (not wet) cloth.
- b. Wipe off or blow away any fluff, dust or dirt on axles or moving parts.

3. Upholstery

- a. Hand-wash (machine washing may damage fabric).
- b. Drip-dry only. DO NOT machine dry as heat will damage fabric.

4. Basic Maintenance Materials available at local store

- a. Auto Wax
- b. Clean rags or cotton cloth.
- c. Teflon-based Lubricant
- d. Mild Detergent.

NOTE– You should not grease or oil the chair.

E. TROUBLESHOOTING CHART

Symptom	Chair veers left or right	Brakes engaged but still loose	Looseness in chair	Sluggish turning	Squeaks and rattles	Caster flutter	Solution:	
	✓	✓	✓	✓				Make sure there is no excessive wear.
			✓	✓	✓	✓		Make sure all nuts and bolts are snug. Do not overtighten or aluminum tubes may be damaged.
			✓	✓	✓			Make sure all spokes are tight on radial spoke wheels.
					✓			Use Tri-Flow Lubricant (Teflon®-based) between all modular frame connections and parts.
	✓					✓		Make sure both front casters touch on a flat area simultaneously. if they do not, they will need to be adjusted by your Premium Retailer

F. TO MOUNT & REMOVE REAR WHEELS 10 11**⚠ WARNING**

Do not use this chair **UNLESS** you are sure both quick-release axles are locked. An unlocked axle may come off during use and cause a fall.

Note: Axle is not locked until:



- The quick-release button (A) pops out fully.
- The ball bearings are fully visible on the outside of the sleeve.

1. To Install Wheel:

- Depress quick-release button (A) fully. This will release tension on ball bearings at other end.
- Insert axle through hub of rear wheel.
- Keep pressure on button (A) as you slide axle (B) into axle sleeve (C)
- Release button to lock axle in sleeve. Have a dealer adjust the axle if it does not lock.
- Repeat these same steps on other side.

2. To Remove Wheel:

- Depress quick-release button (A) fully.
- Remove wheel by sliding axle (B) out of axle sleeve (C).
- Repeat steps on other side.

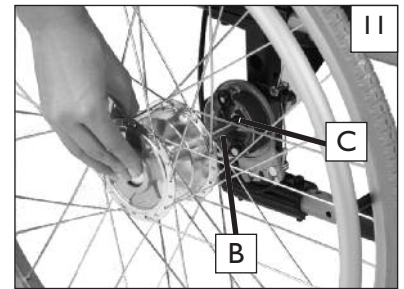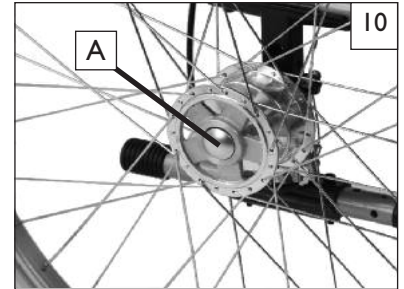**G. WHEEL LOCKS** 12

The Breezy Elegance comes standard with a Push to lock wheel lock.

⚠ WARNING

Wheel locks are installed at Sunrise and should only be adjusted by your qualified service person.

Inspect wheel locks weekly per the maintenance chart. Do not use your chair **UNLESS** you are sure both wheel-locks can fully engage. A wheel-lock that is not correctly adjusted may allow your chair to roll, or turn unexpectedly.

Wheel-locks must be fully engaged, the arm (D) should be imbedded into the tire at least 1/8" to be effective.

If you find the wheel locks have slipped or are not working correctly, contact your service provider for proper adjustment.

**H. DRUM BRAKE (GOLD AND PLATINUM MODELS)** 13 14 15**⚠ WARNING**

Drum Brakes can be used to control speed when an attendant or caregiver is present. The rider of this chair should never attempt to use the Brake handles to stop or slow the chair.

Drum Brakes can also be locked, and used as an extra safety measure for keeping the chair stationary for activities like transfers.

1. Slowing the speed (Attendant)



- Squeeze the two triggers (F) simultaneously to slow the chair and control the speed.

2. Locking the drum brake (Attendant)

- Squeeze the two brake levers and use your index finger to rotate the locking latch (G) into place.
- Release the levers and check to make sure the chair is locked.
- Apply Wheel locks if extra stability is needed.

3. Unlocking the drum brake (Attendant)

- Squeeze the two brake levers, squeezing the trigger hard enough that the locking latch pops out of the lock position.
- You may use the alternate method of squeezing gently and using your index finger to unhook the locking latch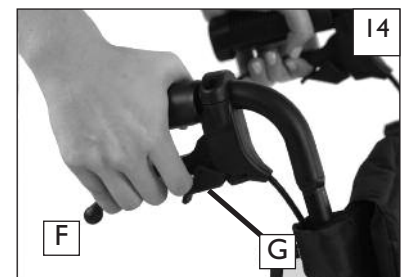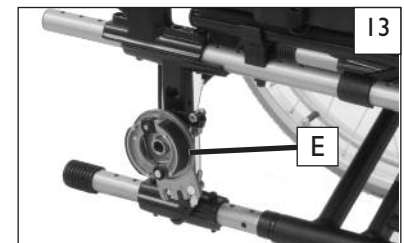

I. ANTI-TIP TUBES (OPTIONAL) 16 17

Sunrise Medical recommends the use of Anti-tip tubes.

1. Inserting Anti-Tip Tubes Into Receiver

- Press the rear anti-tip release button on the anti-tip tube so that both release pins are drawn inside.
- Insert into the anti-tip tube receiver (B).
- Rotate the anti-tip tube until the release pin pops through the receiver mounting hole.
- Insert the second anti-tip tube the same way.

2. Turning Anti-Tip Tubes up.

- Press the rear anti-tip tube release button (A).
- Hold the button in and turn anti-tip tube up.
- Release the button
- Repeat with second anti-tip tube.
- Remember to return anti-tip tubes to down position after completing maneuver.

3. Adjusting Anti-Tip Tube Wheel

The anti-tip tube wheels may have to be raised or lowered to achieve the proper clearance of 1 1/2" to 2".

- Press the anti-tip wheel release button () so that both release pins are drawn inside.
- Raise or lower to one of the three predrilled holes.
- Release button.
- Adjust the second anti-tip tube wheel the same way. Both wheels should be at exactly the same height.

J. FLIP-BACK ARMREST 18

Flip-back armrests are shipped standard on Breezy Elegance wheelchairs, and installed by Sunrise.

- To flip armrests back, depress the release latch (D) and rotate arm toward back of wheelchair.

K. FLIP-BACK HEIGHT ADJUSTABLE ARMREST 19 (GOLD AND PLATINUM MODELS)

Flip-back height-adjustable armrests are installed by Sunrise.

1. Height-Adjustment




- Release the securing lever (2).
- Slide armpad (1) up or down to desired height.
- Move the pad up or down slightly until it is locked in place.

2. Flipping Armrest Back and Removing Armrest

- Depress the release latch (3) and rotate armrest toward the back of wheelchair.
- To remove, pull up on armrest. Pin (6) securing armrest will automatically release.
- To return armrest to operating position, push armpad down until it locks into place.

Flip-Back Height Adjustable

- Armrest pad
- Height adjustment lever
- Flip-back release latch
- Rear receiver
- Armrest panel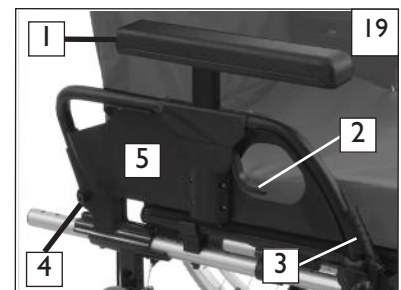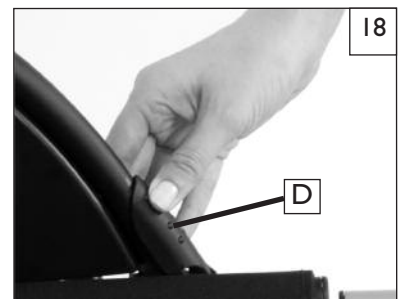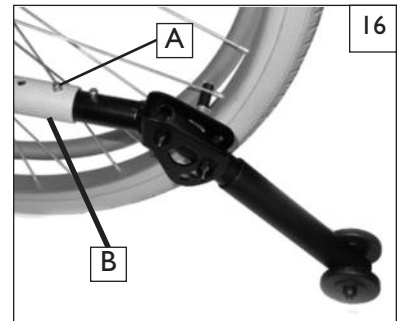

L. SWING-IN/SWING-OUT FOOTREST 20 21

1. Installing the footrest

- Place swing-away pivot saddle (E) into the receiver on front frame tube with the footrest facing outward from the frame.
- Rotate the footrest inward until it locks into place on hanger latch block

2. Removing the footrest

- To remove footrest, pull the release lever toward the frame.
- As you continue to hold the lever, rotate the footrest outward and lift.

M. ELEVATING LEGREST (OPTIONAL) 22**1. Installation**

To install or remove the elevating legrest (ELR) you will use the same basic instructions for swing-away hangers and footrests.

2. Adjustment

- To raise legrest, lift to desired position. Legrest will automatically lock in place.
- To lower, while seated in chair, press release lever forward and lower legrest to desired position. Legrest will automatically lock in place.

**N. BACKREST**


Breezy Elegance wheelchairs are shipped with an 8° bend in the backrest

O. STANDARD OR HEIGHT ADJUSTABLE PUSH HANDLE 23

Your Breezy chair comes standard with push handles. Gold and Platinum models have height adjustable push handles, which can add up to 4" of height adjustment.

1. Adjustment

- To raise or lower the push handles, loosen the height adjustment lever (A) about 1/2 turn and raise or lower the push handles (B) to the desired position.
- Be sure to tighten the adjustment lever when you find the best adjustment.
- Repeat these steps for the other push handle.

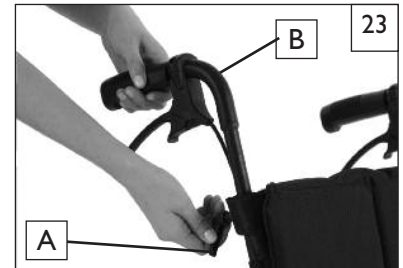**P. CUSHION CARE AND MAINTENANCE** 24**Cleaning your cushion cover**

- Remove the outer cover from the foam base and zip closed.
- Machine wash in warm water and drip or tumble dry on low heat.
- We recommend that you **DO NOT** iron the cover

Cleaning your foam base

- Remove the outer cover from the foam base.
- DO NOT** use soap, wipe lightly with a damp cloth.
- DO NOT** submerge the foam in water;
- Dry off the foam with a clean cloth, and let the foam air dry fully before reassembly.

**Q. UNFOLD OR FOLD YOUR WHEELCHAIR** 25 26

NOTE: For your convenience when storing or transporting your chair, you should remove the quick-release wheels and footrest hangers before you fold the chair, and re-mount them before you unfold chair.

⚠ WARNING

Possible pinch point. DO NOT Fold or unfold your chair if it does not have a seat sling. **Always** use the flat of your palms during unfolding, and push down on the seat rails without wrapping your fingers around the seat rail.

1. Unfolding your Chair:

- Stand to one side of the chair.
- Tilt the chair toward you far enough to take the weight off the opposite wheel.
- Push down on the seat rail closest to you until chair opens fully.
- Make sure seat rail tubes snap into frame saddles, by using the palms of your hand to push down on both seat rail tubes equally.

2. Folding your Chair:

- Make sure footrests are out of the way before you fold the chair. (Swing-away, flip up, or remove the footrests.)
- Remove the cushion, and it is suggested that you remove the rear wheels before transport or storage.
- Grasp center of seat sling and lift it up until chair folds.

R. CHECK-OUT

After the wheelchair is assembled and adjusted, it should roll smoothly and easily. All accessories should also perform smoothly. If you have any problems, follow these procedures.

1. Review the set-up and adjustment section to make sure chair was set up properly.
2. Review the Troubleshooting table in this section.
3. If your problem persists, contact your Premium retailer. If you still have a problem, contact Sunrise customer service. See the introduction page for details on how to contact your Premium Retailer or Sunrise customer service.

⚠ WARNING

The owner of this chair is responsible for making sure that it has been setup and adjusted by a Premium Retailer with advice from a healthcare advisor. Service and/or adjustments should only be done with the advice of your Premium Retailer. Always use parts and/or accessories that have been recommended and approved by Sunrise Medical when servicing this chair.

A. PREMIUM RETAILER SERVICE INTRODUCTION

1. At least once per year, this chair should have a complete inspection, safety check, and regular service made by a Premium retailer.
2. Torque settings—A torque setting is the optimum tightening which should be made on a particular fastener. It is important to use proper torque settings where specified. When not specified, torque settings should be 60 in-lbs
3. If you have discovered a worn, bent, or damaged part, repair or replace them with recommended parts **before returning this chair to service.**
4. All major maintenance and repair work should be done by the Premium Retailer.

B. CRITICAL MAINTENANCE TIPS

1. Torque settings:

A torque setting is the optimum tightening which should be made on a particular fastener. It is important to use proper torque settings where specified. When not specified, torque settings should be 60 in-lbs

2. Premium Retailer and Adjustment Reference materials:

Go to www.sunrisemedical.com for parts manuals, instruction sheets, and instructional videos that will aid in the repair of the Breezy Family of wheelchairs.

C. CLEANING

1. Paint Finish

- a. Clean the painted surfaces with mild soap or detergent.
- b. Protect the paint with a coat of non-abrasive auto wax.

2. Axles and Moving Parts

- a. Clean around axles and moving parts with a slightly damp (not wet) cloth.
- b. Wipe off or blow away any fluff, dust or dirt on axles or moving parts.
- c. **DO NOT USE** 3 in 1 oil, or WD-40 ® for lubrication. Only use Teflon based Lubricant when working on this wheelchair.

D. REAR WHEEL AXLE NUT ADJUSTMENT 28

1. Remove the wheel

- a. The rear wheel axle (C) attaches the rear wheel to the axle plate. When the detent balls (B) engage thru the rear wheel, it effectively becomes locked onto the chair. Pushing the quick-release button (A) disengages the detent balls and allows removal of the axle and wheel assembly.


2. Adjust axle nut

- a. If the wheel and axle will not lock into the axle plate, then the axle requires adjustment. Using a 1/2" open end wrench, turn the nut (D) counter-clockwise approximately 1/4 revolution and try to lock the axle into axle plate. If it doesn't lock, continue making small nut adjustments until it securely locks.
- b. If the wheel is locked on the chair but there is excessive wheel play (the wheel hub can be pushed back and forth on the axle) then adjust the nut clockwise until there is no perceptible gap between the wheel and axle plate, and the axle is securely locked onto the chair.

TOOLS YOU WILL NEED

1. Metric Hex key set
2. Metric Open-end Wrench set
3. Torque wrench
4. Metric Hex bit Socket set
5. Phillips and Flat screwdrivers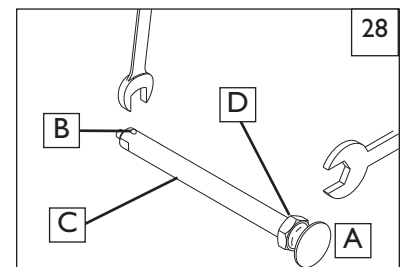


E. VERTICAL AXLE POSITION 29

NOTE: Use a torque setting of 144 in./lbs. to tighten axle nut.

1. Height Adjustment

- a. Remove the rear wheel, to expose the axle plate (E).
- b. Using a Socket and an open end wrench, loosen and remove the axle/drum brake assembly (F).
- c. Move axle up or down to desired position.
- d. Note the hole number and tighten both bolts.
- e. Repeat the process on the opposite side of the wheelchair.
- f. Axle location must be exactly the same on both sides of the wheelchair.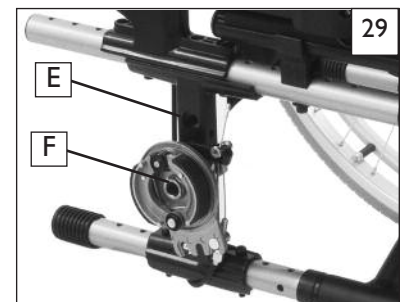


F. SWING-IN/SWING-OUT FOOTRESTS 30 31

IMPORTANT NOTE: These two adjustments might take several tries to get it just right, but the extra time is a great investment in terms of comfort and support for the rider.


1. Lower leg length adjustment

- a. Remove the footrest from the chair before adjustment
- b. Using a 5mm Hex key, loosen the screw (A) that holds the footrest extension arm, and adjust to the appropriate length for the rider to fit comfortably.
- c. Retighten the screw
- d. Repeat these steps on the other footrest.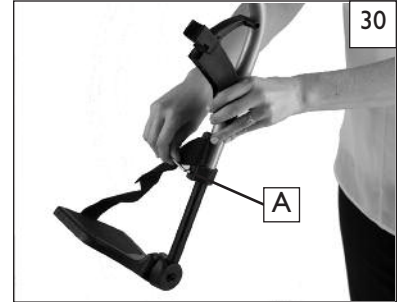

2. Footplate angle adjustment

- a. Remove the footrest, from the chair before adjustment.
- b. Using the 5mm Hex key, loosen the screw (B) that holds the footplate to the extension arm. You are now free to adjust the angle.
- c. Adjust the footplate to the appropriate angle.
- d. Retighten the screw
- e. Repeat these steps on the other footplate
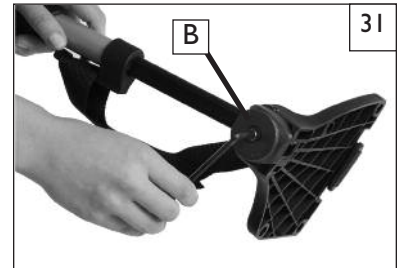


G. CASTER AXLE ANGLE ADJUSTMENT 32 33 34

NOTE: Use a torque setting of 75 in.-lbs. when adjusting caster axle angle.

1. Squaring your caster assembly

- a. Using a 5mm Hex key, remove the dust cover (C), revealing the two attaching screws.(D)
- b. Using the 5mm hex key, loosen two screws(D) just enough so that the caster assembly can be slightly rotated.
- b. Use a large right triangle (E) on a flat surface and the front surface of the caster bearing housing (F). This will align the caster stem rotational axis perpendicular to the flat surface.
- c. When the alignment is good, retighten the two screws (D), and replace the dust cover.
- d. Repeat the process for the other side. making sure that both sides are exactly the same.

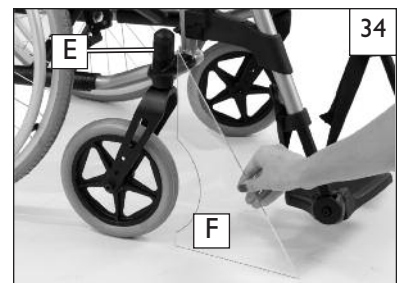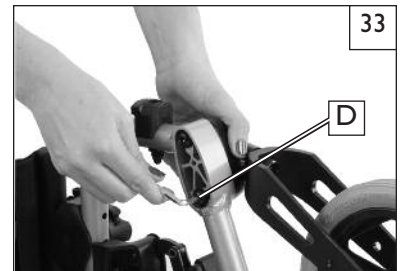H. WHEEL LOCK ADJUSTMENT 35

NOTE: Use a torque setting of 115 in.-lbs. when setting up wheel locks.

1. Adjustment

- a. Loosen the screws (G)on the bottom of each clamp. Do not attempt to remove the screw.
- b. Slide assembly toward rear wheel and tighten the screw.
- c. Check that the wheel lock arm embeds about 1/8" into the tire to prevent wheel movement, when in locked position.
- c. It may take a few tries to get the wheel lock in the right spot.
- d. Repeat the adjustment on the other side, and tighten the screws.

I. LENGTH ADJUSTABLE ARMREST PADS 36 37**Forward or Rearward Placement**

- Using a 5mm hex key, remove both armpad screws (A).
- Move the armrest to the desired location forward (B) or back. (C)
- Replace the screws.
- Check to make sure the adjustment does not remove important support from the fore-arms of the rider.

**J. TENSION ADJUSTABLE UPHOLSTERY** 38 39**! WARNING**

To prevent injury if you are using tension adjustable upholstery, have your Premium Retailer adjust it for you. Loosen straps or remove upholstery prior to folding or unfolding the chair.

The Tension Adjustable Upholstery is designed to follow the contour of the back of the end user and provide beneficial trunk control.

I. Contour Adjustment

- The tension adjustable upholstery is in the rear of your backrest, you can reveal it by pulling up on the back cover (D) as shown
- For a tighter fit, pull the tensioning straps (E) inward to increase the tension and lift the tab of the tension lock buckle to release it.
- You can adjust the upholstery to conform with the user's anatomy. Release the tension on all the straps, and then adjust them one at a time, starting from bottom.
- These adjustments should be done with the user sitting in the chair.

**K. STANDARD AND LOW FRONT SEAT HEIGHTS** 40

The Breezy Elegance wheelchair can be easily changed from standard to low seat heights by making a few simple adjustments.

IMPORTANT NOTE: A front caster angle-adjustment should be made to correspond with any change in seat height.

1. Standard Front Seat Height

To achieve a standard front seat height, position the front casters (F) and rear wheels (G) in the bottom frame and fork settings.

2. Low Front Seat Height

To achieve a lower front seat height, position the front casters and rear wheels in the top frame and fork settings.

**L. SEAT SLING ADJUSTMENT OR REPLACEMENT** 41

The seat sling can be adjusted through the use of VELCRO®-style retaining material beneath the seat.

Seat Sling Tension Adjustment

- Remove the seat rail caps (H) and slide the seat sling out of the rails.
- Readjust VELCRO®-style material to increase the tension in the seat sling. (I).
- Replace the seat sling and re-insert the rail caps.


M. BACK DEPTH ADJUSTMENT

IMPORTANT NOTE:

The Back Depth adjustment is intended to compensate for differences in after-market backrest thickness and/or contour. The back canes can be adjusted about 1" fore and aft relative to the seat so that the backrest support surface does not change the location of the user, regardless of the backrest mounting location. This ensures that the center of gravity of the user relative to the axle location is not altered due to the adjustment for backrest thickness or contour.

I. Back Depth Adjustment

- a. Remove the hardware(A) that holds the backrest canes in place and move them forwards or backwards a maximum of 1".
- b. Ensure that the adjustment is exactly the same on both sides of the chair before re-inserting hardware.
- c. Re-insert the hardware thru the backrest bracket (B) and frame tube (C) and tighten to hold the backcanes firmly in place.

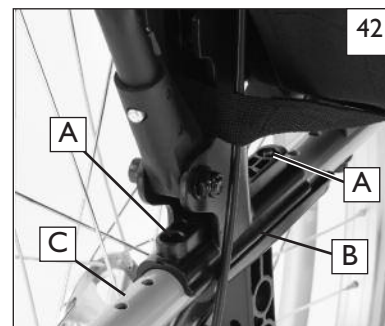N. CHECK-OUT

After the wheelchair is assembled and adjusted, it should roll smoothly and easily. All accessories should also perform smoothly. If you have any problems, follow these procedures.

1. Review the set-up and adjustment section to make sure the chair was set up properly.
2. Review the Troubleshooting table in this section.
3. If your problem persists, contact your dealer or Premium Retailer. If you still have a problem, contact Sunrise customer service. See the introduction page for details on how to contact your Premium Retailer or Sunrise customer service.

A. FOR FIVE (5) YEARS**Frame and cross-brace (if applicable) warranty:**

1. Sunrise guarantees the frame and cross brace against defects in material and workmanship for five (5) years, as long as the original purchaser owns the chair.
2. This warranty does not apply if:
 - a. The chair is subject to abuse
 - b. The chair is not maintained as recommended in the owner's manual
3. The chair is transferred to a different person from the original owner

B. FOR ONE (1) YEAR

We warrant all Sunrise-made parts and components of this wheelchair against defects in materials and workmanship for one year from the date of first consumer purchase.

C. LIMITATIONS

1. We do not warrant:
 - a. Tires, upholstery, pads and push-handle grips.
 - b. Damage from neglect, accident, misuse, or from improper installation or repair.
 - c. Products modified without Sunrise Medical's express written consent.
 - d. Damage from exceeding the weight limit.
2. This warranty is VOID if the original chair serial number tag is removed or altered.
3. This warranty applies in the USA only. Check with your supplier to find out if international warranties apply.
4. This warranty is not transferable and only applies to the first consumer purchase of this wheelchair through your Premium Retailer

D. WHAT WE WILL DO

Our sole liability is to repair or replace covered parts. This is the exclusive remedy for consequential damages.

E. WHAT YOU MUST DO

1. Obtain from your Premium Retailer, while this warranty is in effect, prior approval for return or repair of covered parts.
2. Return the wheelchair or part(s), freight pre-paid, to Sunrise Medical at: 2842 Business Park Avenue, Fresno, CA 93727-1328.
3. Pay the cost of labor to repair, remove or install parts.

F. NOTICE TO CONSUMER

1. If allowed by law, this warranty is in place of any other warranty (written or oral, express or implied, including a warranty of merchantability, or fitness for a particular purpose).
2. This warranty gives you certain legal rights. You may also have other rights that vary from state to state.

Record your serial number here for future reference:

ESCUCHAMOS

Le agradecemos su preferencia por la silla de ruedas Breezy. Nos gustaría recibir sus preguntas o comentarios acerca de este manual, de la seguridad y fiabilidad de su silla, o del servicio que ha recibido de su distribuidor minorista Premium. Thank you for choosing a Breezy wheelchair. We want to hear your questions or comments about this manual, the safety and reliability of your chair, and the service you receive from your Premium Retailer. Please feel free to write or call us at the address and telephone number below:

Sunrise Medical
Customer Service Department
2842 Business Park Ave
Fresno, CA 93727
(800) 333-4000

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©2014 Sunrise Medical (US) LLC